



THE POWER OF **CONNECTED**

January 29, 2018

**URGENT: VOLUNTARY PRODUCT RECALL OF  
EYESALINE EYEWASH AND FIRST AID KITS**

Dear Distributor Partner,

**Reason for Recall:** Honeywell has learned of potential for leaks affecting a limited number of lots of Eyesaline eyewash bottles. While these products are filled aseptically, the potential for a leak results in a low risk of contamination of the eyewash solution and possible damage to surrounding materials when used in First Aid Kits. The root cause has been addressed and we are confident that the issue has been resolved in our production process. Safety is our priority, so we are initiating a voluntary recall of affected products distributors have in inventory, as well as any sold to your customers and end-users.

**Risk to Health:** Although Honeywell has not received and is not aware of any reports of adverse health events related to this issue, exposure to infectious agents due to a compromised container barrier could result in infection and may require treatment with antibiotics.

**Instructions to Customers:**

The lots in question shipped from our distribution center from October 2014 to August 2017. Please take the following actions to help us remove these items from usage:

1. Identify if the product in your possession is affected by locating the lot number on the eyewash bottles or first aid kits and cross-referencing with the list of affected lots you purchased, in the Business Response Form. You can request an electronic copy of this list by contacting 855.215.5028. Reference images 1 through 6 for assistance in locating the lot number.
2. Immediately segregate and stop sales and usage of any affected products.
3. If you have further distributed this product, please notify your customers of this recall at once. All white pages of this notice should be included in the communication to your customers outlining the requested action and should also be posted in your branch. Your customers can obtain replacement eyewash bottles or full first aid kits directly from us by following steps 4 and 5.
4. Create a record of units per lot being returned by filling the enclosed Business Response Form. Use the enclosed return label to return all product in your possession to the addresses below. If necessary, please request additional return labels by contacting 855.215.5028.

Stericycle US	Stericycle Canada
2670 Executive Dr, Suite A	25 Ironside Crescent
Indianapolis, IN 46241	Scarborough(Ontario) M1X 1G5, Canada
5. A copy of the completed Business Response Form must be included with your return.

**LOCATING LOT NUMBERS** - the below images highlight the most common locations for lot number location. Actual location may vary.

**Eyesaline Eyewash Bottles**

Case Label	Bottle	Bottle Close-up
------------	--------	-----------------



Image 1



Image 2



Image 3

**First Aid Kits**

Case Label	First Aid Kit Label	Secondary First Aid Kit Label
------------	---------------------	-------------------------------



Image 4



Image 5

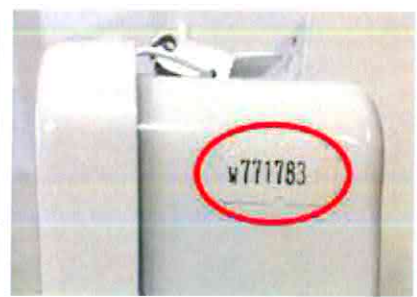


Image 6